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Follow this sequence exactly:

2

- Provide clock with second hand. Note date and time.
- Open container. Remove **one** test strip only. Do not touch reaction zone / test field!
- Close container tightly immediately after removing test strip.

5

- Immerse test strip in the urine for approximately 1 s. The test field must be submerged.
- After removing the test strip from the urine sample, briefly dab the lateral edge on paper. Do not put the test strip down.
- Wait for 30 s.

8

- Start the evaluation: On the container label a color sequence for the test field can be found. Assign the test field to a color value of its sequence. Find a match or select the color value that comes as close as possible to the test field.
- Note the result and submit it to your physician at your next visit.
- Finish the evaluation within 60 s after immersion at the latest. Observe this time limit, since the test field still changes its color intensity after this period.

**Discard test strip after evaluation. DO NOT use more than once!**
**Never** change the intake of prescribed medication as a result of this test.

## Medi-Test Nitrit

### Instructions for Use

**Application**

Screening test for detection of illnesses in the area of the kidneys and the urinary tract.

The test is also suitable for use

- in preventive diagnosis (screening)
- for monitoring progress or relapse during therapy
- for self-monitoring by the patient.

These urine test strips serve exclusively for detecting morbid (pathological) changes in human urine.

**Notes**

Discuss **any positive** or **any unclear** results with your doctor. You must also do this if this test does not give a positive result, but the ailments persist.

If you suspect a false negative result, please repeat the test with a test strip from an unopened container or ask your physician. Your doctor will carry out further investigations with the usual methods in his laboratory. In principle, individual test strip results enable definitive diagnosis and targeted therapy only in combination with other medical findings.

Use only clean, well-rinsed containers to collect urine.

Test urine preferably immediately after collection.

Always remove only the required number of test strips. Do not touch the test fields!

Tightly close container immediately after removal.

Do not remove the drying agent from the stopper, do not damage the stopper.

Protect test strips from sunlight and moisture. Store the container in a cool and dry place (do not refrigerate!) at a temperature between 4 ... 30°C (39 ... 86°F). The test strips can be used up to the imprinted expiry date if stored properly.

Always discard damaged test strips or containers.

The stopper of the test strip container contains a non-toxic drying agent. If it is swallowed accidentally, drink plenty of water (if necessary contact a physician).

## Information and additional notes

**Principle**

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
Microorganisms, which are able to reduce nitrate to nitrite, are indicated indirectly by this test. The principle of Griess reagent is the basis of this test. The test paper contains an amine and a coupling component. A red colored azo compound is formed by diazotisation and subsequent coupling.

**Evaluation – Sources of error**

The test detects concentrations from 0.05 mg nitrite/dL urine. Every pink color indicates a bacterial infection of the urinary tract. The color intensity depends only on the nitrite concentration, but does not provide information about the extent of the infection. A negative result does not preclude an infection of the urinary tract, if bacteria which cannot produce nitrite are present. Falsely negative results can be produced by high doses of ascorbic acid, by antibiotics therapy and by very low nitrate concentrations in urin as the result of low nitrate diet or strong dilution (diuresis). Falsely positive results can be caused by the presence of diagnostic or therapeutic dyes in the urine.

**Quality Control in professional use**

The performance of the test strips should be confirmed by use of positive and negative control solutions. Positive and negative controls should be analyzed once a day, whenever a new bottle of strips is opened, whenever a new lot of strips is started, and every 30 days to check storage conditions. Each laboratory should establish its own goals for adequate standards of performance, and should question handling and testing procedures if these standards are not met.

**MACHERY-NAGEL GmbH & Co. KG** · Neumann-Neander-Str. 6–8 · 52355 Düren · Germany
Tel.: +49 24 21 969-0 · Fax: +49 24 21 969-199 · info@mn-net.com · **www.mn-net.com**

en

IVD

In vitro-diagnostics, suitable for self-testing

## Test strips for rapid detection of nitrite in urine

### Instructions for Use

Dispose used test strips with domestic waste (use test strips only once!).

See below for an explanation of the symbols on box and container label.

Store test strips out of reach of children!

**Explanation of symbols**

Statement of Conformity (Product corresponds to the In-Vitro Diagnostic Medical Devices Directive 98/79/EC of the European Union)

Please read instructions for use!

Permitted storage temperature range

Use by

Batch identification

Item number

Manufacturer

Do not reuse

**MACHERY-NAGEL GmbH & Co. KG** · Neumann-Neander-Str. 6–8 · 52355 Düren · Germany
Tel.: +49 24 21 969-0 · Fax: +49 24 21 969-199 · info@mn-net.com · **www.mn-net.com**

0141/A028728

0197