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

🗑 Discard test strip after evaluation.
DO NOT use more than once!
Never change the intake of prescribed medication as a result of this test.

Test procedure


Follow this sequence exactly: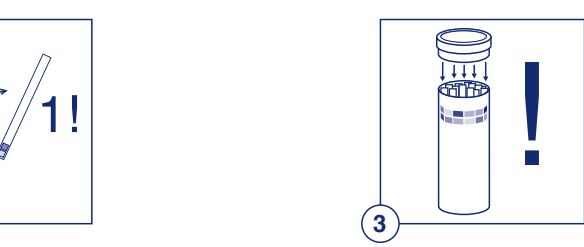

- Provide clock with second hand. Note date and time.
- Open container. Remove **one** test strip only. Do not touch reaction zones/test fields!
- Close container tightly immediately after removing test strip.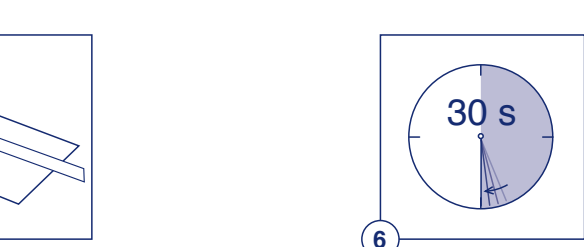

- Immerse test strip in the urine for approximately 1 s. All test fields must be submerged.
- After removing the test strip from the urine sample, briefly dab the lateral edge on paper. Do not put the test strip down.
- Wait for 30 s.

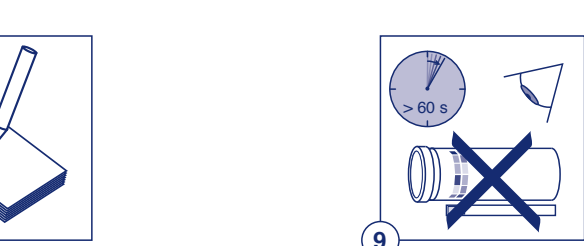- Start the evaluation: On the container label a color sequence for each test field can be found. Assign each test field to a color value of its sequence. Find a match or select the color value that comes as close as possible to the test field.
- Note the result and submit it to your physician at your next visit.
- Finish the evaluation within 60 s after immersion at the latest. Observe this time limit, since some test fields still changes its color intensity after this period.

🗑 Discard test strip after evaluation.
DO NOT use more than once!
Never change the intake of prescribed medication as a result of this test.

Test strips for rapid detection of protein and glucose in urine

Instructions for Use

Application

Screening test for early detection and monitoring of diabetes (diabetes mellitus) and diseases of the kidneys and urinary tract.

The test is also suitable for use

- in preventive diagnosis (screening);
- for monitoring progress or relapse during therapy;
- for self-monitoring by the patient.

These urine test strips serve exclusively for detecting morbid (pathological) changes in human urine.

Notes

Discuss **any positive** or **any unclear** results with your doctor. You must also do this if this test does not give a positive result, but the ailments persist.

If you suspect a false negative result, please repeat the test with a test strip from an unopened container or ask your physician. Your doctor will carry out further investigations with the usual methods in his laboratory. In principle, individual test strip results enable definitive diagnosis and targeted therapy only in combination with other medical findings.

Use only clean, well-rinsed containers to collect urine.

Test urine preferably immediately after collection.

Always remove only the required number of test strips. Do not touch the test fields!

Tightly close container immediately after removal.

Do not remove the drying agent from the stopper, do not damage the stopper.

Protect test strips from sunlight and moisture. Store the container in a cool and dry place (do not refrigerate!) at a temperature between 4–30 °C (39–86 °F). The test strips can be used up to the imprinted expiry date if stored properly.

Always discard damaged test strips or containers.

The stopper of the test strip container contains a non-toxic drying agent. If it is swallowed accidentally, drink plenty of water (if necessary contact a physician).

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Information and additional notes

Principle:

Protein: The test is based on the „protein error“ principle of indicators. The test zone is buffered to a constant pH value and changes colour from yellow to greenish blue in the presence of albumin. Other proteins are indicated with less sensitivity.

Glucose: The detection is based on the glucoseoxidase-peroxidase-chromogen reaction. Apart from glucose, no other compound in urine is known to give a positive reaction.

Evaluation – Sources of error

Protein: The minimum sensitivity of the test strip is 10 mg protein/dL urine. The colour fields correspond to the following ranges of albumin concentrations:

• negative, 30, 100 and 500 mg/dL or negative, 0,3, 1.0 and 5.0 g/L

Falsely positive results are possible in alkaline urine samples (pH > 9), after infusions with polyvinylpyrrolidone (blood substitute), after intake of medicaments containing quinine and also by disinfectant residues in the urine sampling vessel. The protein colouration may be masked by the presence of medical dyes (e.g. methylene blue) or beetroot pigments.

Glucose: Pathological glucose concentrations are indicated by a colour change from green to blue-green. Yellow to weakly green test fields are to be considered as negative (or normal). The colour fields correspond to the following glucose concentrations:

• 0 (negative), 20 (normal), 50, 150, 500 and 1000 mg/dL or

• 0 (negative), 1,1 (normal), 2,8, 8,3, 27,8 and 55,5 mmol/L.

Too low or false negative results are obtained as a result of increased quantities of ascorbic acid, which occur in the urine after a dose of vitamin C (e.g. vitamin pills, antibiotic preparations) as well as after consumption of fruit juice. Gentisic acid also exhibits an inhibiting effect. False positive reactions can be caused by residues of peroxide-containing or other cleansing agents.

Test strips for rapid detection of protein and glucose in urine

Instructions for Use

Application

Screening test for early detection and monitoring of diabetes (diabetes mellitus) and diseases of the kidneys and urinary tract.

The test is also suitable for use

- in preventive diagnosis (screening);
- for monitoring progress or relapse during therapy;
- for self-monitoring by the patient.

These urine test strips serve exclusively for detecting morbid (pathological) changes in human urine.

Explanation of symbols

CE Statement of Conformity (Product corresponds to the In-Vitro Diagnostic Medical Devices Directive 98/79/EC of the European Union)

i Please read instructions for use!

° Permitted storage temperature range

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© 2013 MACHERY-NAGEL GmbH & Co. KG